de PARI NaCl 0,9% Inhalationslösung

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en PARI NaCl 0.9% inhalation solution

Instructions for use

Please read these instructions for use carefully.

IMPORTANT NOTES

Intended purpose: PARI NaCl 0.9% inhalation solution is used to moisten the airways and as a secondary treatment for cold infections. It can also be used as a carrier solution for medications which, according to the manufacturer's instructions, may be diluted with isotonic saline solution for inhalation. PARI NaCl 0.9% inhalation solution is suitable for all age groups and pregnant and nursing women, using a jet, membrane or ultrasonic nebuliser. The dose and duration of use is 2.5 ml to 5 ml up to four times daily as required.

Individuals in the following groups must be supervised by a person who is responsible for their safety:

- Babies, infants, and children
- Individuals with limited capabilities (e.g. physical, mental, sensory)

Indication: Respiratory diseases and to prevent illnesses that can be caused by dry airways.

Contraindications: The manufacturer is not aware of any contraindications.

Side effects: Shortness of breath due to a temporary airway constriction in patients with a hypersensitive bronchial system. In this case, stop the treatment and contact your doctor. No other side effects are known. Inform your doctor and the manufacturer if you notice any side effects that are not listed in the instruction for use.

Interactions: This medication is not known to interact with other medicines.

Safety instructions: The instructions for use contain important information, safety instructions and precautionary measures:

- Read all instructions for use carefully before you start using this PARI product.
- Use this PARI product only as described in the instructions for use.
- Keep the instructions for use in a safe place.
- Always follow the instructions of the inhalation system and nebuliser used.

⚠️ WARNING – may result in death or severe injuries.

- Choking hazard due to small parts which can be swallowed.
- Small parts can block the airways and lead to choking.
- Store all product components out of reach of children.

Danger of condition worsening

If your condition does not improve or your symptoms worsen, seek professional medical advice.

Serious adverse incidents

Report serious adverse incidents to the manufacturer and the responsible competent authority.

Ingredients: Sodium chloride, water; sterile, free from preservatives.

Mode of action: PARI NaCl 0.9% inhalation solution is administered through nebulisation using an inhalation system. The physical mode of action works by moistening the airways. This liquifies the mucus covering the airways. In this way, the clearance mechanism of the lungs is supported. Mucus and other foreign bodies can then be cleared from the airways.

Shelf life: Do not re-use opened ampoules as they may no longer be sterile. Dispose of the inhalation solution after the expiry date.

USE

Use the inhalation solution only at room temperature. Use the ampoule immediately after opening it. Carry out the treatment by following all steps listed below exactly.

Filling the nebuliser with the inhalation solution:

1. Wash your hands thoroughly.
2. Check that the ampoule has not been damaged. Never use an ampoule that has been opened already or damaged, or if the solution is discoloured or cloudy.
3. Open the ampoule by twisting off the upper part.
4. Add the inhalation solution to the nebuliser as described in the nebuliser's instructions for use. Stop the treatment immediately and inform the manufacturer if you notice an unusual odour during inhalation.

FURTHER INFORMATION

Components: As per packaging information.

Disposal: Comply with the applicable disposal regulations of the member country you are in. Always dispose of the product in accordance with the Ordinance on the European Waste Catalogue (EWC).

DOCUMENT INFORMATION

Validity of instructions for use: PARI NaCl 0.9% inhalation solution type 077

Conformity: This product conforms to Medical Device Regulation (EU) 2017/745.

Contact:

Email: info@pari.de

Tel.: +49 (0)8151-279 220 (international)

+49 (0)8151-279 279 (German)

Copyright: ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved.

Trade marks: Registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries: PARI®

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Copyright: ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Technische und optische Änderungen sowie Druckfehler vorbehalten.
Copyright : ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Allemagne. Sous réserve de modifications techniques et visuelles, ainsi que d'erreurs d'impression.
Copyright: ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Alemania. Salvo modificaciones técnicas y de forma y salvo errores de imprenta.
Copyright: ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Germany. Con riserva di modifiche tecniche e visive ed errori di stampa.
Copyright: ©2024 PARI GmbH Spezialisten für effektive Inhalation, Moosstraße 3, 82319 Starnberg, Duitsland. Technische en optische wijzigingen en drukfouten voorbehouden.