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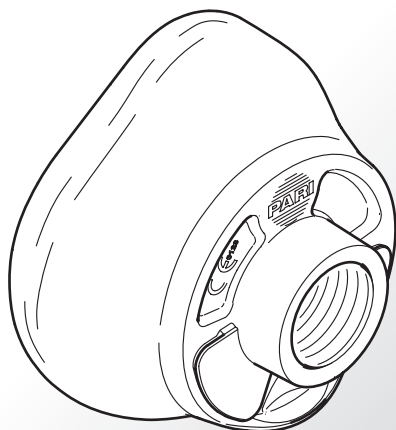
en

Instructions for use

- Read these instructions carefully and completely before using for the first time. Keep these instructions in a safe place.
- The PARI SMARTMASK® Kids is always to be treated as a component or accessory of PARI nebulisers, of which the instructions for use must also be followed accordingly. In particular, the instructions in the chapter entitled "Hygienic re-processing" in these instructions for use also apply for the PARI SMARTMASK® Kids.

Components

Item number: 078G5000 (does not include PARI nebuliser)

Intended use

The PARI SMARTMASK® Kids is an efficient mask inhalation system for children aged 2 and older who are unable or unwilling to inhale using a mouthpiece.

Caution:

To avoid the risk of insufficient oxygen supply, this mask must not be used by children younger than 2 years old.

If the PARI SMARTMASK® Kids is too small, please use the SMARTMASK® for adults.

After the age of 4, the most effective form of treatment for the lower respiratory tract is inhalation with the mouthpiece, because this ensures that the greatest quantity of medication reaches the lungs.

The PARI SMARTMASK® Kids has been designed for repeated use. Its life cycle is 1 year.

The PARI SMARTMASK® Kids must not be used except with the following PARI nebulisers: LC PLUS® family, LC STAR® LL, LC® SPRINT family, eFlow®rapid and a nebuliser based on eFlow® technology.

Use of the mask

- The PARI SMARTMASK® Kids must not be used on children when they are sleeping and/or lying down.
- Make sure that the mask fits snugly in order to minimise aerosol losses.
- Clean and disinfect the mask before it is used for the first time.

Figure A:

- Place the mask on the nebuliser.

Figure B:

- To ensure efficient treatment, hold the nebuliser vertically and the eFlow®rapid nebuliser horizontally.

Figure C:

Make sure that the valves on the mask lie flush against the openings so that the outside surface is smooth! They must not be pressed inwards under any circumstances!

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©2011 PARI Pharma GmbH, 078D1010-D-05/11