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PARI Babymaske soft

PARI Kindermaske soft "Spiggy"

PARI Erwachsenenmaske soft

Model: PARI Maske soft (Type 041)

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info@pari.de • www.pari.com

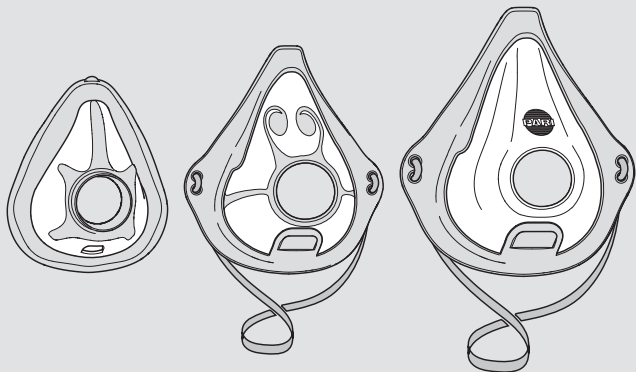
Instructions for use

PARI baby mask soft
PARI child mask soft "Spiggy"
PARI adult mask soft

Model: PARI Maske soft (Type 041)

Masks for PARI inhalation systems

Read the instructions for use

Read these instructions carefully before using the product. Follow all instructions and safety directions. Keep the instructions in a safe place.

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Moosstrasse 3, 82319 Starnberg, Germany. Technical and design changes and printing errors reserved. The same applies to illustrations.

Disclaimer

These instructions for use describe the components of PARI products and optional accessories. For this reason, these instructions for use also describe and illustrate features not present in your PARI product because they are, for instance, country-specific and/or optional. When using the systems, products and functions, the applicable country-specific regulations must be observed.

Trade marks

Registered trade marks of PARI GmbH Spezialisten für effektive Inhalation in Germany and/or other countries:

LC SPRINT®, PARI®, VORTEX®

TABLE OF CONTENTS

IMPORTANT INFORMATION	4
Intended purpose.....	4
Indication	5
Contraindication.....	5
Labelling	5
Safety and warning instructions.....	6
PRODUCT DESCRIPTION	8
Components	8
Overview and designations	8
Product variants.....	8
Product combinations	9
Description of function	9
Material information	10
Service life	10
USE	11
Preparing for treatment.....	11
Performing treatment.....	13
Ending the treatment	15
REPROCESSING	16
Reprocessing cycles.....	17
Limits of reprocessing.....	19
Preparation for reprocessing	19
Reprocessing of the mask	19
Further information about reprocessing.....	23
FURTHER INFORMATION	23

1 IMPORTANT INFORMATION

1.1 Intended purpose

The PARI mask soft is an accessory for inhalation treatment. That enables inhalation of aerosol¹ through the mouth and nose.

The different mask sizes are suitable for treating patients in the following age groups:

- PARI baby mask soft: Babies aged from 6 months to 3 years
- PARI child mask soft "Spiggy": Children aged 4 years and above
- PARI adult mask soft: Adults

The specified ages are approximate. The actual size of the mask depends on the size and shape of the person's face.

The mask must be used only with PARI nebulisers.

The PARI adult mask soft can also be used with the "VORTEX" holding chamber.

This PARI product can be used in a home environment, as well as in professional health institutions. When used in a home environment, this PARI product is intended for single-patient use only (no patient change). In a professional environment, the device can be used with different patients as long as the corresponding hygiene reprocessing measures are complied with.

This product must be used only by individuals who understand the contents of the instructions for use and are able to use the product safely.

Individuals in the following groups must be supervised by a person who is responsible for their safety:

- Babies, infants, and children
- Individuals with limited capabilities (e.g. physical, mental, sensory)

If the patient is not able to use this product safely on their own, then the treatment must be carried out by the responsible person.

This PARI product is suitable only for patients who are able to breathe by themselves and are conscious.

1) Aerosol: Small particles of solid, liquid or mixed composition (fine "mist") suspended in gases or air.

1.2 Indication

For patients who cannot inhale using a mouthpiece, or if inhalation via mask is preferred. The mask forms a system with a nebuliser or a holding chamber. The indication for this system is the same as the indication for the nebuliser or holding chamber used.

1.3 Contraindication

There are no contraindications known to PARI GmbH.

1.4 Labelling

The following symbols can be found on the product and/or the packaging:

	Medical device
	Legal manufacturer
	Date of manufacture
	This product conforms to the EU Medical Device Regulation 2017/745.
	Item no.
	Production batch number, lot number
	Unique Device Identifier (UDI)
	Consult instructions for use
	Elastic band
	Mask connector
	Mask stabiliser
	PARI baby mask soft
	PARI child mask soft "Spiggy"
	PARI adult mask soft

1.5 Safety and warning instructions

The present instructions for use contain important information, safety instructions and precautionary measures. The user must follow these in order to guarantee safe operation of this PARI product.

This PARI product must be used only as described in these instructions for use.

The instructions for use of the compressor and nebuliser used or the holding chamber as applicable must also be followed.

Labelling and classification of warning instructions

In these instructions for use, safety-critical warnings are categorised according to the following hazard levels:

DANGER

DANGER indicates a hazardous situation which will lead to very severe injuries or death if it is not avoided.

WARNING

WARNING indicates a hazardous situation which can lead to very severe injuries or death if it is not avoided.

CAUTION

CAUTION indicates a hazardous situation which can lead to mild or moderate injuries if it is not avoided.

NOTE

NOTE indicates a hazardous situation which can lead to material damage if it is not avoided.

General

If your health condition does not improve or it even worsens as a result of the treatment, seek professional medical advice.²

Skin care products containing oils or fats can damage the soft plastic components. The patient should refrain from using skin care products of this kind while using the device.

2) Professional medical staff: Doctors, pharmacists, and physiotherapists.

Hazard due to small parts which can be swallowed

The product contains small parts. Small parts can block the airways and lead to a choking hazard. Keep all components of the product out of the reach of babies and infants at all times.

Treatment of babies, infants, and anyone who requires assistance

For individuals who are not able to perform the therapy session without assistance or cannot appreciate the hazards, the risk of injury is greater. Such individuals include for example babies, children and people with limited capabilities. For these individuals, a person responsible for safety must supervise or carry out the application.

Impairment of the therapy by disregarding the reprocessing instructions

CAUTION

Impairment of the therapy

Exceeding the reprocessing instructions can result in damage to the product. This can impair the therapy.

Falling short of the instructions can result in insufficient reprocessing. This can increase the risk of infection.

- Comply with the instructions about the reprocessing limits, temperature, holding time, and concentration of the chemical used.

Reporting serious incidents

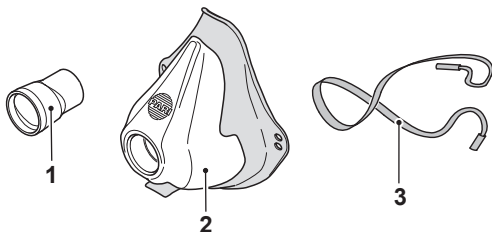
Report serious adverse incidents to the manufacturer and to the competent authority.

2 PRODUCT DESCRIPTION

2.1 Components

Please refer to the package for information on the supplied components.

2.2 Overview and designations

1	Mask connector
2	Mask
3	Elastic band

2.3 Product variants

The PARI mask soft is available in three sizes:

Mask type	Age group
PARI baby mask soft	Babies from 6 months to 3 years
PARI child mask soft "Spiggy"	Children aged 4 years and above
PARI adult mask soft	Adults

The mask type depends on the size and shape of the face. The ages shown are intended as guideline values.

2.4 Product combinations

The PARI masks can be combined with the following PARI products:

PARI baby mask soft PARI child mask soft "Spiggy"	All PARI nebulisers (except PARI LC SPRINT SINUS, PARI LC SPRINT Tracheo, eFlow rapid, Velox and PARI BOY free)
PARI adult mask soft	– All PARI nebulisers (except PARI LC SPRINT SINUS, PARI LC SPRINT Tracheo, eFlow rapid, Velox and PARI BOY free) – VORTEX (with mask connector)

2.5 Description of function

The PARI mask soft is an accessory for PARI inhalation systems. The PARI adult mask soft is also an accessory for the PARI VORTEX holding chamber.

The PARI mask soft allows aerosol to be inhaled through the mouth and the nose.

The patient can breathe out through the aperture or the exhalation valve at the bottom end of the mask without having to take the mask off.

The PARI child and adult mask soft can be fixed to the face using the elastic band. The elastic band is attached to the loops on the side of the mask.

Only in professional healthcare institutions is the mask stabiliser inserted into the mask during the sterilisation process, to maintain the shape of the mask.

The mask connector functions as a connecting piece between the PARI adult mask soft and the VORTEX.

2.6 Material information

The individual product components are made from the following materials:

Product component	Material
PARI mask soft	Polypropylene, thermoplastic elastomer
Elastic band	Synthetic rubber
Mask connector	Polypropylene

2.7 Service life

The individual product components have the following expected lifetimes:

Product component	Service life
PARI mask soft, mask connector	Home environment [see: Limits of reprocessing in a home environment, page 19]
PARI mask soft, mask connector	Professional environment [see: Limits of reprocessing in a professional healthcare institution, page 19]

3 USE

All the steps described below must be carried out properly.

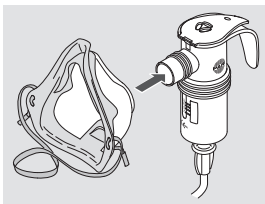
Do not use product components unless they have been thoroughly cleaned and dried. Wash your hands thoroughly before every use. You must perform cleaning and disinfection before using the device for the first time.

3.1 Preparing for treatment


Prepare for an inhalation session via mask as follows:

With a PARI nebuliser

- Assemble the nebuliser as described in the accompanying instructions for use.
- Attach the mask to the nebuliser.

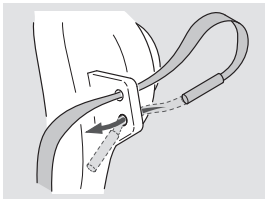
- If necessary, attach the elastic band to the mask.


With a VORTEX

Prepare for an inhalation session via mask as follows:

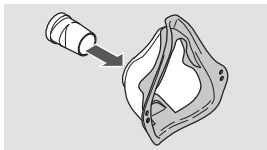- Assemble the VORTEX as described in the accompanying instructions for use.
- Take the protective cap off the mouthpiece of the VORTEX.
- Insert the mask connector in the opening in the PARI adult mask soft.

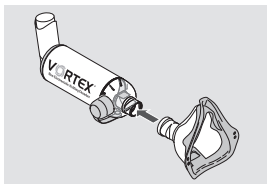- Attach the mask to the mouthpiece of the VORTEX with the mask connector.

3.2 Performing treatment

All the safety instructions and warnings in these instructions for use must have been read and understood before any treatment is carried out.

With a PARI nebuliser

INHALING WITH A MASK

CAUTION

Impaired treatment due to escaping aerosol

If the mask does not form a seal on the face, aerosol may escape. This may result in medication underdosage.

- Make sure that the mask completely covers both corners of the mouth and the nose.
- Take note of possible side effects caused by escaping aerosol. These are described in the information for use of the respective medication.

To carry out an inhalation session via mask, proceed as follows:

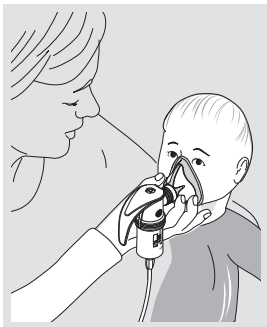
- Fill the nebuliser according to the appropriate instructions or use.
- Carry out the generally applicable steps listed in the section "Performing treatment" in the instructions for use of the nebuliser you are using.

With PARI baby mask soft

-  **WARNING! Risk of suffocation!** If the nebuliser is tilted by more than 70°, the inhalation solution can run through the mask and into the child's mouth and nose, resulting in a risk of suffocation. Therefore the child should not lie down while inhaling. Gently press the mask snugly over the mouth and nose. Make sure that the hole at the lower end of the mask is unobstructed and that the nebuliser is vertical.

Tip for easier use:

Children may struggle when the mask is pressed against their face, twisting their head back and forth. To ensure effective inhalation, hold the mask firmly in your hand and rest your little finger against the child's cheek. This will enable you to follow the movements of the child's head with the mask more easily. Make sure that the nebuliser is always upright.


- The child should breathe in as slowly and deeply as possible through the mask and out again calmly.
- Carry out the inhalation treatment until no more aerosol is generated.


💡 *Some residual fluid will remain in the nebuliser after the end of the treatment.*

With PARI child or adult mask soft

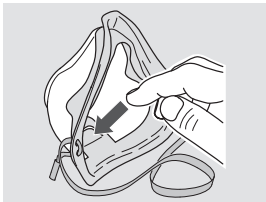- Check whether the exhalation valve is pressed outwards, to ensure that the user can breathe out freely during the inhalation session.

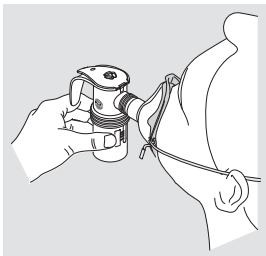- Sit in an upright position and relax.
- Use gentle pressure to place the mask tightly over mouth and nose. Ensure that the nebuliser is positioned vertically.
- If necessary, use the elastic band to hold the mask firmly in place against the face. The elastic band runs along the back of the head.

- Breathe in as slowly and deeply as possible through the mask, and out again calmly.
- Carry out the inhalation treatment until no more aerosol is generated.

 *Some residual fluid will remain in the nebuliser after the end of the treatment.*

With a VORTEX

Treatment with the PARI adult mask soft in combination with the VORTEX is described in the instructions for use of the VORTEX.

3.3 Ending the treatment

End the treatment as described in the instructions for use of the nebuliser you are using.

4 REPROCESSING

CAUTION

Risk of infection due to cross-contamination in the case of a change in patients

If a product is used for more than one patient, there is a risk that germs may be transmitted from one patient to the next.

- Clean, disinfect and sterilise all separated parts before every patient change.

CAUTION

Risk of infection due to growth of bacteria

Inadequate disinfection encourages the growth of bacteria and thus increases the risk of infection.

- During disinfection, comply with the specified holding time and the concentration of the chemical used.
- When using a disinfectant, ensure that the device is clean and operating properly. Allow the disinfection to continue until the disinfectant switches off automatically or the minimum disinfection time stated in the instructions for use of the disinfectant has elapsed. Do not switch the device off prematurely.

CAUTION

Risk of infection by residual germs

If there is dirt on the parts, germs capable of reproduction may remain despite the sterilisation process. As a result, there is a danger of infection.

- Clean, disinfect, and dry all parts thoroughly before sterilising.
- Use only validated procedures for cleaning and disinfection.

CAUTION

Risk of infection due to moisture

Moisture encourages the growth of bacteria.

- Ensure that the components are allowed to dry sufficiently after each processing step.

NOTE

Risk of damage to plastic parts!

Plastic melts if it comes into contact with hot surfaces.

- During thermal disinfection, you must ensure that the water level in the container is sufficient, in order that the individual parts do not come into contact with hot surfaces.

Always use drinking water for reprocessing in a home environment.

 *In professional health care institutions, drinking water is sufficient for precleaning. For all other reprocessing steps and processes, use deionised water with a low microbiological load (at least drinking water quality).*

Please observe the instructions for use for the chemicals used.

Wash your hands thoroughly before every reprocessing.

4.1 Reprocessing cycles

Perform the reprocessing steps as per the specified reprocessing cycles.

Reprocessing cycles in a home environment

Mask components	– Clean immediately after every use – Disinfect once per week ³
Elastic band	Cleaning visible dirt

3) For the therapy of patients at risk, disinfect the individual parts once per day. Take note of the additional information for patients at risk [see: Reprocessing for patients at risk, page 18].

Reprocessing cycles in professional healthcare institution

BEFORE A CHANGE OF PATIENTS

Mask components	– Cleaning – Disinfection – Sterilisation
Elastic band	Replacing the elastic band

SINGLE PATIENT USE

Mask components	– Clean immediately after every use – Disinfect once per week ³
Elastic band	Cleaning visible dirt

Reprocessing for patients at risk

CAUTION

Danger of infection for patients at risk

For patients at risk, airway infections represent an increased risk of deterioration of general health, because these patients are particularly vulnerable to residual germs. Patients at risk include cystic fibrosis patients, patients with immunosuppression or immunodeficiency, and vulnerable patient groups.

- If you are a patient at risk, disinfect the individual parts once per day during the therapy.
- If you are unsure whether you are a patient at risk, consult with specialist medical personnel before use.

4.2 Limits of reprocessing

Limits of reprocessing in a home environment

Mask and mask connector, disinfection	300 processing cycles, max. 1 year
---------------------------------------	------------------------------------

Limits of reprocessing in a professional healthcare institution

Mask and mask connector, disinfection	300 processing cycles, max. 1 year
Mask and mask connector, sterilisation	100 processing cycles, max. 1 year

4.3 Preparation for reprocessing

Procedure

Mask

- Disconnect all mask components from the nebuliser and the VORTEX.
- If necessary, detach the mask connector from the mask.
- If necessary, detach the elastic band from the mask.

4.4 Reprocessing of the mask

The following products can be cleaned, disinfected and sterilised according to the procedure described below:

– PARI mask soft

The elastic band of the mask must be treated separately.

Procedure

Step 1: Precleaning

All individual parts must be precleaned immediately after use.

EQUIPMENT:

– Drinking water temperature of about 15 °C

PROCEDURE:

- Rinse all parts used for 2 minutes in running drinking water.

Procedure

Home environment

Step 2: Cleaning

EQUIPMENT:

- Drinking water temperature of about 40 °C
- Standard commercial washing-up liquid
- Container having sufficient capacity

PROCEDURE:

- Unless otherwise specified by the manufacturer of the washing-up liquid, add about 1 teaspoon of washing-up liquid to 3 l warm drinking water.

- Place all the parts in the washing-up water.
Application time: 5 minutes
- Occasionally move the parts back and forth.
- In case of visible soiling, use a medium-soft brush (e.g. a toothbrush) which is used exclusively for this purpose.

RINSING:

- Rinse all individual parts thoroughly under running drinking water at approx. 15 °C for 3 minutes.

Cleaning the elastic band

- Clean the elastic band as necessary with warm drinking water and a little dishwashing liquid.

The elastic band cannot be disinfected or sterilised.

Procedure

Home environment	Step 3: Disinfection	<p>A - Thermal disinfection using boiling water</p> <p>EQUIPMENT:</p> <ul style="list-style-type: none"> – Clean cooking pot – Drinking water <p>PROCEDURE:</p> <ul style="list-style-type: none"> • Place all the individual parts in water at a rolling boil for at least 5 minutes. <hr/> <p>B - Using a standard thermal disinfectant for baby bottles</p> <p>EQUIPMENT:</p> <ul style="list-style-type: none"> – Thermal disinfectant with a runtime of at least 6 minutes. <p>PROCEDURE:</p> <p>Regarding the performance of the disinfection, the duration of the disinfection procedure and the quantity of water required, follow the instructions for use of the disinfectant you are using.</p>
In professional health care institutions	Step 2: Cleaning and disinfection	<p>EQUIPMENT:</p> <ul style="list-style-type: none"> – Neodisher® MediClean forte (concentration: 0.5%) – Deionised water – Cleaning and disinfection device as per ISO 15883. <p>PROCEDURE:</p> <ul style="list-style-type: none"> • Programme for cleaning and thermal disinfection (at least A0 = 3000) as per manufacturer's instructions.

Procedure

In professional health care institutions	Step 3: Sterilisation	<p>For sterilisation, use a process compliant with the ISO 17665 series of standards.</p> <p>PROCEDURE:</p> <p>Sterilisation temperature and holding time:</p> <ul style="list-style-type: none"> – Temperature: 134 °C – Holding time: minimum 3 minutes to maximum 5 minutes 	
		<p>Information on sterilising a PARI mask soft:</p> <p>Always use the corresponding mask stabiliser when sterilising this mask type, because otherwise the mask may lose its shape under the effects of high temperatures.</p>	
		<ul style="list-style-type: none"> • Insert the mask stabiliser in the mask as shown in the illustration. • Pack all the disassembled parts in a sterile barrier system according to ISO 11607-1 (e.g. foil-paper packaging). • Carry out the sterilisation in a steam steriliser in accordance with the manufacturer's instructions. 	
Drying	<p>Ensure that there is no residual moisture remaining in the components. If necessary:</p> <ul style="list-style-type: none"> • Shake the water out of all of the parts. • Place all parts on a dry, clean and absorbent surface and allow them to dry completely. • Using a standard thermal disinfecter for baby bottles: Leave all individual parts in the closed thermal disinfecter for max. 24 hours until the next use. 		
Visual inspection	<p>Inspect all product components after each cleaning, disinfection or, where applicable, sterilisation. Replace defective, deformed or seriously discoloured parts.</p>		
Storage	<p>Store this product as described below:</p> <ul style="list-style-type: none"> – Dry and dust-free, e.g. in a clean, lint-free cloth (e.g. a tea towel) – Protected against contamination, where necessary (e.g. using optional sterile packaging) 		

4.5 Further information about reprocessing

Further validated processes for reprocessing

The instructions provided were validated by PARI and were found to be suitable for preparing your medical device for its reuse.

Further validated processes for reprocessing:

https://www.pari.com/fileadmin/041D0624_Professional_healthcare_institution_Validated_Reprocessing_Methods.pdf

Ensure that the reprocessing actually performed by your personnel on your equipment with the chemicals used achieves the desired results. For this, validation and routine monitoring of the process are usually required. In particular, if you have to deviate from our validated process, ensure that the reprocessing method selected by you is appropriately effective, and that potential adverse effects are assessed.

5 FURTHER INFORMATION

5.1 Disposal

All product components may be disposed of with normal domestic waste. The country-specific disposal regulations must be observed.

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Copyright