



## PARI VORTEX®

Antistatic Holding Chamber/Spacer

WITH NEW PEDIATRIC MASKS



[www.pari.com/vortex](http://www.pari.com/vortex)

# Advantages of PARI VORTEX®



- Antistatic, aluminum chamber ensures efficient and consistent medication delivery with MDIs for optimal treatments.
- The flexible connecting ring is compatible with most common metered-dose inhalers.
- Mouthpiece cap protects the chamber from unwanted debris.
- PARI VORTEX® grows with you—our interchangeable masks eliminate the need to purchase a new spacer when changes in mask size are needed.
- The optional patented one-handed operation aid makes treatments easier with babies or for those who have manual coordination difficulties.

## PARI VORTEX® – ideal for patients of all ages



**VORTEX with Pediatric Mask, Sm**  
0 - 1.5 years  
Part No.: 051F8100  
HCPCS No.: A4627



**VORTEX with Pediatric Mask, Med**  
1.5 - 4 years  
Part No.: 051F8200  
HCPCS No.: A4627



**VORTEX with Adult Mask**  
4 years and up  
Part No.: 051F8300  
HCPCS No.: A4627



**VORTEX Pediatric Mask, Sm**  
0 - 1.5 years  
Part No.: 044F8100



**VORTEX Pediatric Mask, Med**  
1.5 - 4 years  
Part No.: 044F8200



**VORTEX with Mouthpiece**  
4 years and up  
Part No.: 051F8000  
HCPCS No.: A4627



**One-Handed Operation Aid**  
Part No.: 044F5100

## PARI VORTEX® PEDIATRIC MASKS RECEIVED A 4.8/5 STAR CHILD ACCEPTANCE RATING



### EASY CLEANING AND DISINFECTION

Thanks to the durable aluminum chamber and removable mouthpiece, PARI VORTEX® and masks can be easily cleaned and disinfected.

You can clean the PARI VORTEX® and accessories with lukewarm water and liquid dish soap, or by placing all parts in the top rack of the dishwasher. To easily disinfect, place all parts in a clean pot of boiling water for 5 minutes.

### PARI LENDS A HAND WITH THE ONE-HANDED OPERATION AID

The patented one-handed operation aid helps simplify treatments with babies and small children as well as for adults who may have manual coordination issues or reduced strength in their fingers.



# VORTEX® pediatric masks



## Excellent aerosol performance

Robust seal even when applied gently to the face



## Comfortable fit

The soft mask fits seamlessly to the contours of the face



## High level of patient acceptance

Ease of use and comfortable fit make it a patient and caregiver favorite



## Dishwasher safe

Easy to clean, dishwasher safe, and boilable



## Small

0 - 1.5 years


## Medium

1.5 - 4 years



## Environmentally friendly

Not made with BPA, latex, or phthalates

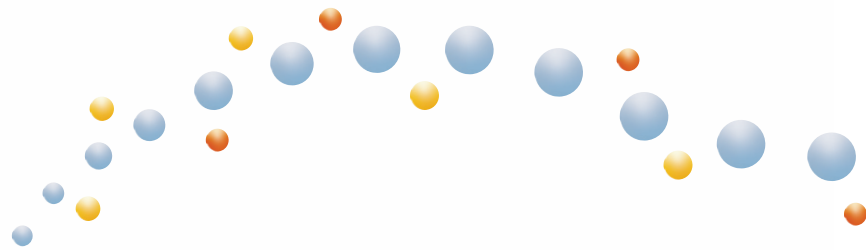

## Aluminum Chamber. Less Plastic. Better Delivery.



**PARI** Respiratory Equipment, Inc.  
Tel: 1-800-FAST-NEB (327-8632)  
product.info@pari.com  
www.pari.com | Midlothian, VA 23112