

PARI lends a hand with the **one-handed operation aid**



The patented one-handed operation aid helps parents and caregivers deliver treatments to babies using just one hand, so you can hold your child safely during inhalation.

Scan the QR code below for a short video on how to use the PARI VORTEX® and accessories



## PARI VORTEX® for patients of all ages



**VORTEX with Pediatric Mask, Sm**  
0 - 1.5 years



**VORTEX with Pediatric Mask, Med**  
1.5 - 4 years



**VORTEX with mouthpiece**  
4 years and up

### VORTEX® baby and child masks— comfortable and efficient

- **A comfortable fit** – The soft mask fits seamlessly to the contours of the face
- **Efficient inhalation** – Robust seal even when applied gently to the face
- **Super easy hygiene control** – Easy to clean, dishwasher safe, and boilable



**PARI** Respiratory Equipment, Inc.  
2412 PARI Way, Midlothian, VA 23112  
Tel: 1-800-FAST-NEB (327-8632)  
product.info@pari.com | www.pari.com



## PARI VORTEX® Antistatic Holding Chamber

### QUICK GUIDE

Aluminum Chamber.  
Less Plastic.  
Better Delivery.



www.pari.com/vortex

## As easy as 1, 2, 3: Assembly & inhalation with... mouthpiece



1 Remove the inhaler cap, shake vigorously



2 Insert the inhaler into the receptacle on the back of the holding chamber



3 Remove the protective cap on the mouthpiece



4 Exhale, then place the mouthpiece between teeth and close lips around it



5 Press down to actuate the inhaler



6 Breathe in and out slowly and deeply

**Did your physician prescribe several sprays per use?** Repeat steps 5 and 6.

**0 - 18 months:** 4 - 6 breaths, **>18 months:** 2 - 4 breaths

## mask and one-handed operation aid



1 Remove the inhaler cap, shake vigorously



2 Push the one-handed operation aid onto the holding chamber



3 Fit the inhaler into the connecting ring and click the one-handed operation aid into place



4 Remove the protective cap and attach the mask



5 Apply to face over nose and mouth creating a good seal



6 Actuate the inhaler, breathe in and out slowly and deeply

**Did your physician prescribe several sprays per use?** Repeat step 6.

**0 - 18 months:** 4 - 6 breaths, **>18 months:** 2 - 4 breaths

## Cleaning and disinfection: Step-by-step in the video

Thanks to the durable aluminum chamber and removable mouthpiece, the VORTEX® and masks can be boiled, disinfected, and washed in the dishwasher.

